

INSTRUCTIONS FOR USE

IFU CODE : 06



E12
4 Prong Socket
Adapter W/Center
Hole



E13
4 Prong Socket
Adapter Large



INFORMATION



Last Update: 01-12-2022

- Please read this document carefully before using the product.
- Instruct the user in the appropriate and safe use of the product.
- Follow the safety instructions to abstain injuries and damage to the product.
- Please keep this document in a safe place.

Contents

1.E12: 4 Prong Socket Adapter W/Center Hole	1
2.E13: 4 Prong Socket Adapter Large	1
3.Assembly	2
4.Weight Limit of Products.....	2
5.Environmental Conditions.....	2
6.Service Life	3
7.General Safety Instructions	3
8.Maintenance	3
9.Warranty	4

1.E12: 4 Prong Socket Adapter W/Center Hole

For direct lamination onto BK socket.

With pyramid hole through pyramid and lamination anchor.

All socket adapters comes with its cushion dummy.



1.1 Properties

Code	Material	Weight
E 12s	Stainless Steel	90 gr
E 12s -134 kg		110 gr
E 12T	Titanium	66 gr
E 12Al	Aluminum	71 gr

2.E13: 4 Prong Socket Adapter Large

For direct lamination onto BK socket.

With pyramid hole trough pyramid and lamination anchor.

Adapter diameter is 72 mm.



2.1 Properties

Code	Material	Weight
E 13	Stainless Steel	108 gr

3.Assembly

4-Prong socket adapters have designed to be laminated on the BK Sockets. Stockinette and woven carbon fibers should be prepared on the socket prior to putting the lamination anchor. Place the lamination anchor at the socket end as required for proper alignment. Plastic dummy of the product should be putted on the lamination anchor to keep the pyramid zone clean. Stabilize and fix the anchors with sealing resin/talcum powder mixture. Pull required amount of stockinette and carbon fibers over the model. Complete the lamination process with the resin. Once the resin is cured, remove the plastic dummy.

▲CAUTION!

Do not bend the anchor arms of the product.

4.Weight Limit of Products

Weight Limit	Product Code	
100 kg	E 12s	E 12AI
125 kg	E 13s	
134 kg	E 12s -134 kg	E 12T

5.Environmental Conditions

Acceptable Environmental Conditions

Temperature range: -10 °C to +45°C

Moisture: Allowable relative humidity 20 % to 90 %, non-condensing

Unsuitable Environmental Conditions

Chemicals/liquids: Salt water, perspiration, urine, acids e.g.

Solids: Sand, dust in high concentrations, highly hygroscopic particles (e.g., talcum)

6. Service Life

The service life of the product is about 3 years. This duration depends on the daily activity of the user.

7. General Safety Instructions

▲ CAUTION!

To avoid the risk of injury and product damage, do not use for longer than the service life of the product.

Avoid using this product in unsuitable environmental conditions. If the product has been exposed to unsuitable environmental conditions, check for damage.

Do not use the product that is damaged, has limited functionality or is in doubtful condition. Ensuring that appropriate measures are taken (e.g. control, cleaning, repair, replacement by the authorized workshop or manufacturer).

To prevent mechanical damage, be careful and caution when working the product.

Use the product for a single patient to avoid injuries and product damage.

Products should be used in appropriate combinations. Using the product with unsuitable combinations may result in possible injury.

In case of feeling any space or sound, apply the rehabilitation center who installed the foot.

8. Maintenance

1. After the first 30 days of use, the prosthesis parts should be visually inspected and function checked.
2. Inspect the entire prosthesis for wear during normal consultations.
3. Annual security checks should be implemented.

9.Warranty

IB-ER company provides the warranty for the products in case of material and production flaws. Warranty duration for the above-mentioned products are individually mentioned at below.

Parts	Duration after the IB-ER Invoice Date
E 12s	24 Months
E 12s -134 kg	
E 12T	
E 12Al	
E 13	

This warranty covers on only replacement and repair of failed products after assemblies performed by qualified individuals and competent institutions. This warranty is not applicable in case of following points:

- Improper use of the product, using with unsuitable conditions and fault assembly.
- Issues arising from replacement of our products with other products.



İB-ER Makina A.Ş
Selvitepe OSB Mah. OSB 2006. Cad. No: 13/1
45400, Turgutlu - Manisa / Turkey

Phone : +90 (236) 312 32 00

Fax : +90 (236) 312 32 06

info@ib-er.com

www.ib-er.com



Ercan Gozuacik
Reinickendorfer Str 45 13347 Berlin GERMANY
(for regulatory affairs only)