INSTRUCTIONS FOR USE

IFU CODE: 14

IB-ER
PROSTHETIC

Single Axis Friction Knee

(E 42, E 42-EXT)

CE

Fig. 1

Abb. 1.


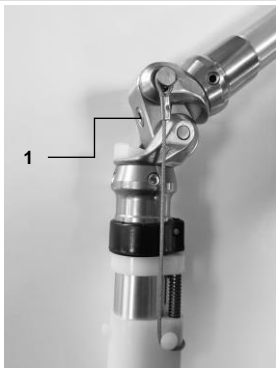
Fig. 2

Abb. 2.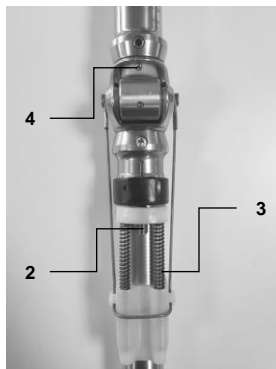

▲ ATTENTION!

- Please read this document carefully before using the product.
- Instruct the user in the appropriate and safe use of the product.
- Follow the safety instructions to abstain injuries and damage to the product.
- Please keep this document in a safe place.

Last Update 25.01.2024

EN Instructions for use	1
DE Gebrauchsanweisung	6
FR Instructions d'utilisation	12
CE - Declaration of Conformity	18

1. E 42 – Single Axis Friction Knee

This is a uniaxial and compact knee joint, specially produced for the users requiring high degree of stability. It's has a friction ability under the stance phase which provides the stability. Swing phase speeds can be adjusted with the extension assist and friction adjustment. This knee joint should be used together with its extension assist.

1.1. Features

Code	Material	Weight	Weight Limit
E 42s	Stainless Steel	485 gr	100 Kg
E 42T	Titanium	376 gr	100 Kg

2. E 42 – Extension Assist

This is a set of the extension assist parts of the E42 knee joint. It provides the extension phase to be smooth and adjustable. This set includes one plastic cover, one adjustment ring, two compression springs, one distal anchor shell and one wire bow.

2.1. Features

Code	Weight
E 42-EXT	118 gr

3. Adjustment of Knee

CAUTION!

E42 Single Axis Friction Knee is factory adjusted!
The prosthesis is ready to be installed!

As a first step, try on the user without changing or playing the adjustment!

If further adjustment needed, then proceed to the following steps:

3.1 Adjusting the Stance Phase Control

The responsiveness of the brake effect may be regulated by the stance control adjustment screw (see number 1 in fig. 1).

Clockwise rotation= braking is less responsive

Counter-clockwise rotation= braking is more responsive

3.2 Adjusting Swing Phase with Extension Assist

By turning the adjustment screw (see number 2 in fig. 2) the tension of the extension spring (see number 3 in fig. 2) can be adjusted.

Clockwise rotation= increases tension

Counter-clockwise rotation= decreases tension

3.3 Setting Swing Phase Speed with the Friction Adjustment

Resistance to movement between the knee axis and the brake bushing may be adjusted with the set screw (see number 4 in fig. 2).

Clockwise rotation= movement slower

Counter-clockwise rotation= movement faster

With small movements of the screw, the changes will be perceptible. Those which are too tight will cause the brake to be activated even when not under load. Settings which are too loose lead to early abrasion and may cause noise.

4. Environmental Conditions

Acceptable Environmental Conditions
Temperature use range: -10 °C to +45°C
Humidity: Allowable relative humidity 20% to 90%, non-condensing

Unsuitable Environmental Conditions
Fresh water, salt water, sweat, urine, acids e.g.
Sand, dust, highly hygroscopic particles (e.g., talcum)
Mechanical vibrations, impacts
Cleaning agents containing solvents

5. Service Life

The service life of the product is about 3 years. This duration depends on the daily activity of the user.

6. General Safety Instructions

CAUTION!

To avoid the risk of injury and product damage, do not use for longer than the service life of the product.

The suitability of the device to the patient and the prosthesis should be evaluated by a healthcare professional.

The device must be fitted and adjusted by a healthcare professional.

There is a danger of pinching and injuring your finger or skin.

Do not insert your hand into the joint mechanism!

Avoid using this product in unsuitable environmental conditions. If the product has been exposed to unsuitable environmental conditions, check for damage.

Do not use the product that is damaged, has limited functionality or is in doubtful condition. Ensuring that appropriate measures are taken (e.g. control, cleaning, repair, replacement by the authorized workshop or manufacturer).

To prevent mechanical damage, be careful and caution when working the product.

Use the product for a single patient to avoid injuries and product damage.

Products should be used in appropriate combinations. Using the product with unsuitable combinations may result in possible injury.

In case of feeling any space or sound, apply the rehabilitation center who installed the foot.

7. Maintenance

1. After the first 30 days of use, the prosthesis parts should be visually inspected and function checked.
2. Inspect the entire prosthesis for wear during normal consultations.
3. This prosthesis and components should be checked for its functions at least once a year with regular intervals by a prosthesis specialist. There is no risk if suggestions are followed by properly.

8. Warranty

IB-ER company provides the warranty for the products in case of material and production flaws. Warranty duration for the above-mentioned products are individually mentioned at below.

Product	Duration After The IB-ER Invoice Date
E 42	24 Months

This warranty covers on only replacement and repair of failed products after assemblies performed by qualified individuals and competent institutions. This warranty is not applicable in case of following points:

- Improper use of the product, using with unsuitable conditions and fault assembly.
- Issues arising from replacement of our products with other products.

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We hereby certify and declare our sole responsibility for the following Medical Device(s) which relates to *Class 1* according to Medical Directives MDR 2017/745.

E42s	Single Axis Friction Knee, Stainless Steel	UDI-DI: 8680757251485
E42T	Single Axis Friction Knee, Titanium	UDI-DI: 8680757251492
E42-EXT	E42 Friction Knee Extension Kit	UDI-DI: 8680757251508

The above mentioned products are in conformity with the following standards or other normative documents:

- **ISO 10328** Structural Testing of Lower-limb Prostheses
- **ISO 14971** Risk Analysis
- **MDR 2017/745** Regulation of European Parliament 2017/745

Authorized Representative in EC

Ercan Gözüacık
Reinickendorfer Str 45
13347 Berlin
GERMANY

Erol ERYILMAZ
General Manager
İB-ER MAKİNA A.Ş.

This declaration is valid from January 01, 2024

İB-ER Makina A.Ş
Selvilitepe OSB Mah. OSB 2006. Cad. No: 13/1
45400, Turgutlu - Manisa / Turkey

Phone : +90 (236) 312 32 00

Fax : +90 (236) 312 32 06

info@ib-er.com

www.ib-er.com

Ercan Gozuacik
Reinickendorfer Str 45 13347 Berlin GERMANY
(for regulatory affairs only)