INSTRUCTIONS FOR USE

IFU CODE: 17


IB-ER
PROSTHETIC

Rotation Adapter
Product Code: ROT

CE

Fig 1.

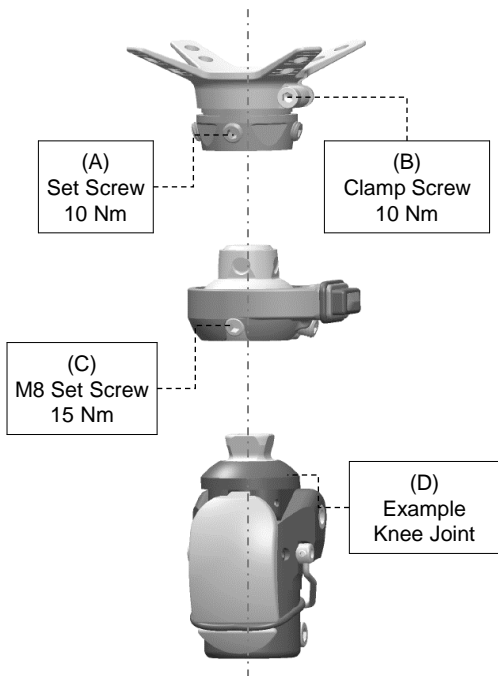English

Last Update: 27.08.2024

- Please read this document carefully before using the product.
- Instruct the user in the appropriate and safe use of the product.
- Follow the safety instructions to abstain injuries and damage to the product.
- Please keep this document in a safe place.

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1. Description

Rotation adapter connects the knee joint with the proximal part of the prosthesis. It allows the knee joint to turn inward and outward. The ROT can rotate 360 degrees.

When it is in the zero position, it snaps into place automatically.

This adapter significantly improves comfort of the user by rotating the prosthesis when needed. Daily activities like getting in and out of the car, tying shoelaces or putting on socks are made easier and more comfortable.

Code	Material	Weight Limit
ROT	Stainless Steel	125 kg

2. Usage

1. Press the button while the prosthesis not loaded to enable rotation.
2. Rotate the device as needed. It can be rotated 360°.
3. The device is automatically locked when it is returned to the starting position.

ATTENTION!

Only load the rotation adapter in the locked state. Make sure the rotation adapter is in the locked position before loading the prosthesis. Loading an unlocked rotation adapter can cause injuries and falls.

**CAUTION!**

Be sure that the locking system snaps back into place during use.

3. Assembly

Tighten **M8 Set Screws (C)** using torque wrench.

Torque value: **15 Nm**

**ATTENTION!**

Secure screw thread with medium strength adhesive. Always clean the setscrew teeth before each assembly.

**CAUTION!**

Mounting the pyramid part of the rotation adapter

Tighten **Set Screws (A)** using torque wrench.

Torque value: **10 Nm**

Secure screw thread with medium strength adhesive.

After tightening the setscrews to be mounted on the pyramid part of the rotation adapter to the recommended torque, check the operation of the rotation adapter before the adhesive applied to the bolt threads dries. The rotation adapter should operate silently and linearly as it did before the setscrews were tightened. There should be no jamming during rotation movement.

4. Environmental Conditions

Acceptable Environmental Conditions
Temperature use range: -10 °C to +45°C
Humidity: Allowable relative humidity 20% to 90%, non-condensing

Unsuitable Environmental Conditions
Fresh water, salt water, sweat, urine, acids e.g.
Sand, dust, highly hygroscopic particles (e.g., talcum)
Mechanical vibrations, impacts

5. Service Life

The service life of the product is about 3 years. This duration depends on the daily activity of the user.

6. General Safety Instructions

CAUTION!
 To avoid the risk of injury and product damage, do not use for longer than the service life of the product.
The suitability of the Rotation Adapter to the patient and the prosthesis should be evaluated by a healthcare professional.
Avoid using this product in unsuitable environmental conditions. If the product has been exposed to unsuitable environmental conditions, check for damage.
Do not use the product that is damaged, has limited functionality or is in doubtful condition. Ensuring that appropriate measures are

taken (e.g. control, cleaning, repair, replacement by the authorized workshop or manufacturer).

To prevent damage, be careful and caution when working the product.

Use the product for a single patient to avoid injuries and product damage.

Products should be used in appropriate combinations. Using the product with unsuitable combinations may result in possible injury.

In case of feeling any space or sound, apply the rehabilitation center who installed the prosthetic.

7. Maintenance

1. After the first 30 days of use, the prosthesis parts should be visually inspected and function checked.
2. Inspect the entire prosthesis for wear during normal consultations.
3. This prosthesis and components should be checked for its functions at least once a year with regular intervals by a prosthesis specialist. There is no risk if suggestions are followed by properly.

8. Warranty

IB-ER company provides the warranty for the product in case of material and production flaws. Warranty duration for the above-mentioned product is mentioned at below.

Product	Duration After The IB-ER Invoice Date
ROT	12 Months

This warranty covers on only replacement and repair of failed products after assemblies performed by qualified individuals and competent institutions. This warranty is not applicable in case of following points:

- Improper use of the product, using with unsuitable conditions and fault assembly.
- Issues arising from replacement of our products with other products.

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