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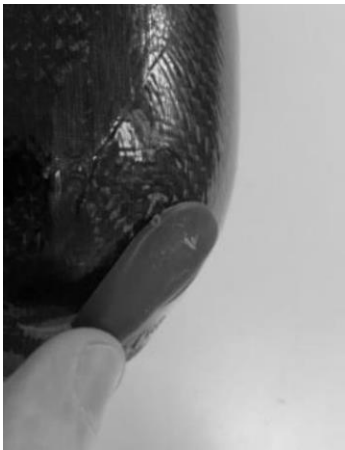
INFORMATION

Last Update: 27-02-2023

- Please read this document carefully before using the product.
- Instruct the user in the appropriate and safe use of the product.
- Follow the safety instructions to abstain injuries and damage to the product.
- Please keep this document in a safe place.

Contents

1. Description and Functions.....	1
2. Features	1
3. Installation	1
4. Care & Cleaning	3
5. Environmental Conditions	3
6. Service Life.....	3
7. Scope of Delivery.....	3
8. General Safety Instructions.....	4
9. Warranty	4

1. Description and Functions

The Vac-100 adapter has been specially created to provide a vacuum inside of the prosthetic socket, to increase the grabbing and prevent any pistoning movement.

Vac-100 adapters are active vacuum adapters, which generate a vacuum with every step of the user.

It generates a vacuum up to -0.5 Atm vacuum pressure within only 8-10 steps of the user.

2. Features

Connection Type: Pyramid Receiver & Tube Clamp

Code	Weight Limit of Products	Tube Diameter	Material	Product
VAC-100-30	100 Kg	30	7075-T6 Aluminum	

3. Installation

The top part of the adapter is a regular pyramid receiver section to be easily attached to the pyramid section of a socket adapter.

The bottom part is a dia. 30 mm tube clamp section, to be attached to a tube adapter.

To discharge the air and provide vacuum inside of the socket, the socket should be drilled with the size of the “black socket connector” that comes together with the Vac-100 product (Fig. 2).

The hose of the connector should be positioned inside of the socket.

The hose of the connector and the vacuum hose can be cut to any desired length. It's important not to shorten it too much, this might

cause extra tension, instead the slightly loose hose can be wrapped around the prosthesis.

Once the connector is positioned inside the socket, it should be glued to the socket, like in Fig. 3.


It's important to slightly grind the surface around the head cap of the socket connector, to increase the grab for the glue.

 **Important Tip 1**

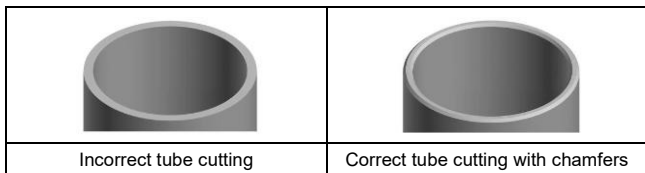The Vac-100 Vacuum adapter comes in totally adjusted position and vacuum value is set by the fabrication (≈ -0.5 Atm). Therefore, it doesn't require any special adjustment to provide vacuum pressure.

 **Important Tip 2**

After cutting the Tube adapter to be fitted with the knee joint, it is important to make "chamfers" inside and outside of the tube!

- It's important to clear the surface of the tube after the cutting & deburring.
- Clean the threads before every installation.
- While tightening the M6 tube clamp screw, apply the specified torque value, **15 Nm** (See Fig.1).
- For the final assembly, secure the screws with the preferred Loctite Threadlocker series.
- Do not lubricate or grease the adapter.

4. Care & Cleaning

The vacuum hose should be de-attached at least every 3 months and should be gently cleaned with water. Any visible particle in the hose or socket connector (on the socket), should be removed to provide a stable airflow.

Important Tip 3

Inside of the hose and connector can be cleaned with a compressor gun (with air) to remove particles.

5. Environmental Conditions

Acceptable Environmental Conditions

Temperature use range: -10 °C to +45°C

Humidity: Allowable relative humidity 20% to 90%, non-condensing

Unsuitable Environmental Conditions

Fresh Water, salt water, sweat, urine, acids e.g.

Sand, dust, highly hygroscopic particles (e.g., talcum)

Cleaning agents containing solvents

Mechanical vibrations, impacts

CAUTION!

Do not expose the product to unallowable environmental conditions.

6. Service Life

The service life of the product is about 3 years. This duration depends on the daily activity of the user.

7. Scope of Delivery

Title	Quantity	Reference Number
Instructions for use	1	Code 21
Active Vacuum Unit	1	VAC-100
Vacuum Hose Ø 6 x 770 mm t: 1 mm	1	VAC100-16
Plastic Socket Connector	2	VAC100-09

Screws are included in the Vacuum Unit.

8. General Safety Instructions

 CAUTION!
To avoid the risk of injury and product damage, do not use for longer than the service life of the product.
Avoid using this product in unsuitable environmental conditions. If the product has been exposed to unsuitable environmental conditions, check for damage.
Do not use the product that is damaged, has limited functionality or is in doubtful condition. Ensuring that appropriate measures are taken (e.g. control, cleaning, repair, replacement by the authorized workshop or manufacturer).
To prevent mechanical damage, be careful and caution when processing the product.
Use the product for a single patient to avoid injuries and product damage.
Products should be used in appropriate combinations. Using the product with unsuitable combinations may result in possible injury.
In case of feeling any space or sound, contact the rehabilitation center who installed the foot.

9. Warranty

IB-ER company provides the warranty for the product in case of material and production flaws. Warranty duration for the above-mentioned product is mentioned below.

Product	Duration After The IB-ER Invoice Date
VAC-100-30	18 Months

This warranty covers on only replacement and repair of failed product after assemblies performed by qualified individuals and competent institutions. This warranty is not applicable in case of following points:

- Improper use of the product, using with unsuitable conditions and fault assembly.
- Issues arising from replacement of our product with other products.

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