

INSTRUCTIONS FOR USE

IFU CODE: 36

IB-ER
PROSTHETIC



E 35T-3P
E 35s-3P



E 36T-3P
E 36s-3P



E 35T-4P
E 35s-4P



E 36T-4P
E 36s-4P




E 35T-P
E 35s-P



E 36T-P
E 36s-P

Fig 1. (E 35T-3P or E 35s-3P)


Fixing
Screw
13 Nm

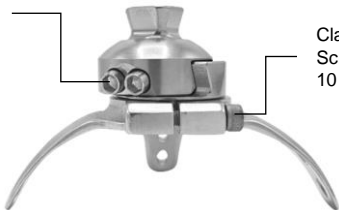

Clamping
Screw
10 Nm

Fig 2. (E 35T-4P or E 35s-4P)

Fixing
Screw
13 Nm

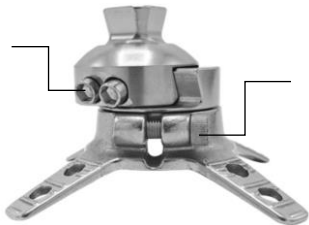

Clamping
Screw
10 Nm

Fig 3. (E 36T-3P or E 36s-3P)

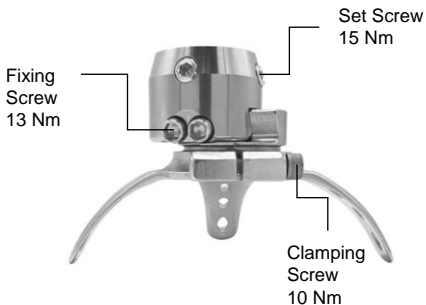


Fig 4. (E 36T-4P or E 36s-4P)

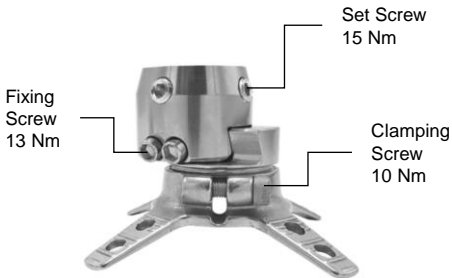


Fig 5. (E 35T-P or E 35s-P)


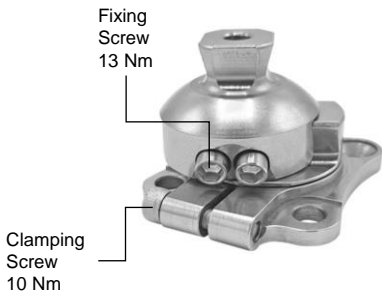


Fig 6. (E 36T-P or E 36s-P)



INFORMATION

Last Update: 04.06.2024

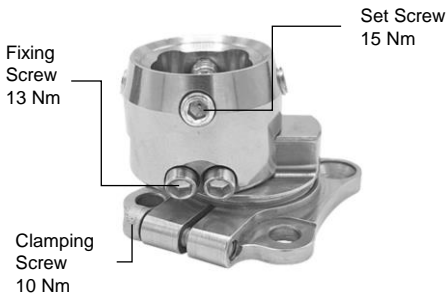

- Please read this document carefully before using the product.
- Instruct the user in the appropriate and safe use of the product.
- Follow the safety instructions to abstain injuries and damage to the product.
- Please keep this document in a safe place.

Contents

1. 3 and 4 Prong Socket Adapters with Sliding and Rotating Combined.....	1
2. E04 series Plate with Rotatable & Sliding Adjustment Adapter	2
3. Assembly	2
4. Environmental Conditions.....	3
5. Service Life	3
6. General Safety Instructions	3
7. Maintenance	4
8. Warranty	4

1. 3 and 4 Prong Socket Adapters with Sliding and Rotating Combined

These products are the combinations of E35 and E36 series, together with the prong adapters

Allows easy adjustment by rotating and sliding at the same time

Capable to rotate 360° and slide 11 mm at maximum range

Thanks to its short length, allows easy alignment and saves extra space

1.1. Features

Code	Material	Weight	Weight Limit
E 35T-3P	Titanium / S. Steel	220 gr	100 Kg
E 35T-4P		190 gr	
E 36T-3P		250 gr	
E 36T-4P		220 gr	

Code	Material	Weight	Weight Limit
E 35s-3P	S. Steel / S. Steel	295 gr	100 Kg
E 35s-4P		265 gr	
E 36s-3P		340 gr	
E 36s-4P		310 gr	

2. E04 series Plate with Rotatable & Sliding Adjustment Adapter



E 04s-Plate



E 35s-P
E 35T P



E 36s-P
E 36T P

- Ready-made combinations with E04 series Plate:

Code	Material	Weight	Weight Limit
E 35s-P	Stainless Steel	251 gr	100 Kg
E 36s-P		296 gr	
E 35T-P	Titanium	153 gr	
E 36T-P		310 gr	

3. Assembly

A torque wrench should be used to tighten the screws. The force in the figures must be applied to adjust the switch. (See fig. 1, 2, 3, 4, 5, 6)

CAUTION!



Use a thread locker to secure all threaded fasteners.

Make sure that the bolt connections are assembled without any errors.

Clean the bolt threads thoroughly before each assembly.
Observe the recommended bolt tightening torque values.

4. Environmental Conditions


Acceptable Environmental Conditions
Temperature use range: -10 °C to +45°C
Humidity: Allowable relative humidity 20% to 90%, non-condensing

Unsuitable Environmental Conditions
Fresh water, salt water, sweat, urine, acids e.g.
Sand, dust, highly hygroscopic particles (e.g., talcum)
Mechanical vibrations, impacts

5. Service Life

The service life of the product is about 3 years. This duration depends on the daily activity of the user.

6. General Safety Instructions

CAUTION!
 To avoid the risk of injury and product damage, do not use for longer than the service life of the product.
The suitability of the device to the patient and the prosthesis should be evaluated by a healthcare professional.
The device must be fitted and adjusted by a healthcare professional.

Avoid using this product in unsuitable environmental conditions. If the product has been exposed to unsuitable environmental conditions, check for damage.

Do not use the product that is damaged, has limited functionality or is in doubtful condition. Ensuring that appropriate measures are taken (e.g. control, cleaning, repair, replacement by the authorized workshop or manufacturer).

To prevent mechanical damage, be careful and caution when working the product.

Use the product for a single patient to avoid injuries and product damage.

Products should be used in appropriate combinations. Using the product with unsuitable combinations may result in possible injury.

In case of feeling any space or sound, apply the rehabilitation center who installed the foot.

7. Maintenance

1. After the first 30 days of use, the prosthesis parts should be visually inspected and function checked.
2. Inspect the entire prosthesis for wear during normal consultations.
3. This prosthesis and components should be checked for its functions at least once a year with regular intervals by a prosthesis specialist. There is no risk if suggestions are followed by properly.

8. Warranty

IB-ER company provides the warranty for the products in case of material and production flaws. Warranty duration

for the above-mentioned products are individually mentioned at below.

Product		Duration After The IB-ER Invoice Date
E 35T-3P	E 35s-3P	24 Months
E 35T-4P	E 35s-4P	
E 36T-3P	E 36s-3P	
E 36T-4P	E 36s-4P	
E 35s-P	E 36s-P	

This warranty covers on only replacement and repair of failed products after assemblies performed by qualified individuals and competent institutions. This warranty is not applicable in case of following points:

- Improper use of the product, using with unsuitable conditions and fault assembly.
- Issues arising from replacement of our products with other products.



İB-ER Makina A.Ş
Selvilitepe OSB Mah. OSB 2006. Cad. No: 13/1
45400, Turgutlu - Manisa / Turkey

Phone : +90 (236) 312 32 00

Fax : +90 (236) 312 32 06

info@ib-er.com

www.ib-er.com



Ercan Gozuacik
Reinickendorfer Str 45 13347 Berlin GERMANY
(for regulatory affairs only)